

Techno-Aide, Inc. 7117 Centennial Boulevard Nashville, TN 37209

P: 800.251.2629 F: 800.554.6275

MOBILE RADIATION BARRIERS CLEANING & CARE INSTRUCTIONS 2018

Thank you for choosing Techno-Aide for your radiation protection needs. To ensure the longevity of your new mobile barrier, we recommend the following guidelines for cleaning and care.

CLEANING:

Window is made with high lead oxide content glass; the surface is susceptible to scratches and attack by acids and alkalis. Use only water, non-abrasive cleaning agents and a soft cloth. Always dry the glass. Barrier frame may be cleaned with any non-Ammonia based or non-chlorine based cleaner. Any neutral based disinfecting product may be used for disinfecting if necessary. Products such as Neutral Q, Century Q, DC Gold etc. DO NOT USE Ammonia, Bleach or other Acid or Alkali based cleaners.

STORAGE: When not in use, barrier should be stored in area free of loose objects that may accidentally fall and break glass. Caster should be locked to prevent unintentional movement. Normal room temperature and low relative humidity is ideal.

<u>MAINTENANCE</u>: Examine barrier frequently. If window become pitted, chipped or in other ways damaged cease use immediately and contact your distributor for instructions. No end user maintenance required or recommended other than cleaning.

RECYCLING: Leaded glass contains 70% lead oxide, frames are powder coated steel and a small amount of plastic. Frames can be recycled with normal metal or plastic. Windows should be sent to an approved lead recycling facility.